WHITE[®] | BEAUTY PROFESSIONAL

Genius Pro

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ULTRA-FAST PROFESSIONAL WHITENING TREATMENT

White Beauty Professional GENIUS PRO is an ultra-rapid professional whitening system powered by 6% hydrogen peroxide. White Beauty Professional GENIUS PRO can be applied after professional oral hygiene or, in some cases, as a stand-alone ultra-rapid whitening treatment.

White Beauty Professional GENIUS PRO does not require the use of gingival dam and can be used with or without LED light.

HOW TO OBTAIN THE BLUE GEL SOLUTION READY FOR USE

- Before proceeding with mixing, wear safety devices (gloves, mask and glasses) and protect the patient.
- Remove the GENIUS PRO syringe from the kit and unscrew the cap and insert the connector (Luer Lock).
- Unscrew the cap of GENIUS POWDER and connect the syringe to the connector to link them together.
- Once both syringes have been connected to the connector, vigorously press the plunger of the GENIUS PRO syringe, completely pouring the contents of the liquid syringe into the GENIUS POWDER syringe.
- Once the liquid has been poured into the GENIUS POWDER syringe, remove the adhesive seal on the GENIUS PRO syringe. Apply backstops.
- Continue mixing the product by pressing the syringe pistons alternately and vigorously 3/4 times.
- Transport the product from one syringe to another in order to obtain a homogeneous blue gel. Once a homogeneous gel has been obtained, transfer all the product obtained into the GENIUS PRO syringe alone and remove the connector.
- Insert the tip and gently compress the blue gel solution to let the air out of the syringe.

PROCEDURE

Before treatment, it is advisable to carry out a professional oral hygiene session on the patient or to check that their teeth are correctly cleaned.

After inserting a mouth opener, mix the two syringes and apply the freshly mixed gel on the teeth you want to treat (the use of a gum dam is not necessary).

Leave the product to act for 10 minutes, suctioning the patient's saliva. Remove any product residues using the saliva ejector by removing the upper cap to facilitate suction. If the professional deems it appropriate, a second application can be carried out.

The use of the LED lamp is recommended to accelerate the teeth whitening process and its use remains at the discretion of the professional.

WARNINGS:

Contains hydrogen peroxide (6%).

Limit unnecessary personal contact; avoid contact with eyes; wear adequate protective equipment.

In case of contact with eyes, rinse immediately and abundantly with water. If irritation persists, consult your doctor.

Store in a cool, well-ventilated area, away from heat, sparks, flames and sunlight.

PRODUCT FOR PROFESSIONAL USE - The syringe is disposable and can be used by the professional on a single patient. Not to be used on a person under 18 years of age. The use of the product is at the discretion of the professional following the clinical evaluation.

APPLICATION

METHOD TO BE PERFORMED ONLY AT THE DENTAL OFFICE

Before to start the whitening treatment procedure, the patient should be informed about the complete procedure and has to sign the informed consent.

The professional in the office takes the patient's medical history and notes the indication for treatment via the color scale or the WBP App. Make sure that the teeth to be treated are absolutely clean and proceed with the color setting.

In case of proven dentin sensitivity, the preventive application of the treatment based on micronized hydroxyapatite is recommended.


Before mixing the blue gel product in the syringe, wear gloves, a mask and protective glasses.

Remove the GENIUS PRO syringe from the package, unscrew the cap and insert the connector.

Unscrew the cap of GENIUS POWDER and connect the syringe to the connector.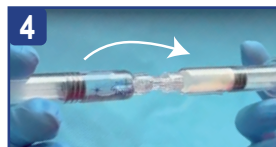

Once both syringes have been connected to the connector, vigorously press the piston of the GENIUS PRO syringe, completely pouring the liquid into the GENIUS POWDER syringe.

Continue mixing the product by pressing the syringe pistons alternately and vigorously 3/4 times.

Transport the product from one syringe to another in order to obtain a homogeneous blue gel. Once a homogeneous gel has been obtained, transfer all the product obtained into the GENIUS PRO syringe alone and remove the connector.

Insert the tip and compress the homogeneous blue gel until the air inside the syringe escapes.

Apply the contents of the syringe to the teeth you wish to treat. Leave the product to act by suctioning the patient's saliva.

The duration (10/15 minutes per step) and the number of treatments (max 2) remains the professional's decision based on the patient's clinical assessment.

The use of the lamp is at the discretion of the professional.

Remove any product residues using the saliva ejector by removing the top cap to facilitate exhalation.

Perform color sampling to check the results obtained.

STORAGE: Store the product at room temperature, protecting the package from heat and exposure to direct sunlight.

The expiry date refers to the perfectly preserved product. Do not use after the expiry date shown on the package.

USAGE NOTES:

- For patients who react with excessive tooth sensitivity, decrease contact time.
- Check the condition of your teeth and gums during treatment.
- Once the two syringes have been mixed, the product can be used for a maximum of 60-90 minutes.

MIROMED GROUP SA
Corso San Gottardo 54A - 6830 Chiasso - SWITZERLAND || www.miromedgroup.ch
MIROMED S.r.l. - Viale De Gasperi 17 - Lainate (MI) ITALY || info@miromed.it