WHITE[®] | BEAUTY PROFESSIONAL

Tailor

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WHITE[®] | BEAUTY PROFESSIONAL

Tailor

PROFESSIONAL HOME WHITENING 10 or 16% carbamide peroxide based for trays and aligners

White Beauty Professional TAILOR is an innovative professional home whitening system that creates a "tailor-made" carbamide peroxide-based whitening gel for each patient that can be used with orthodontic aligners, tray boils and bites, orthodontic retainers and with whitening trays with reservoirs. The whitening gel is prepared by the professional directly in the dental office before being delivered to the patient: this type of preparation improves the action of the carbamide peroxide, making it more effective over time. The whitening gel, immediately after preparation, is placed in the White Beauty Professional TAILOR Airless dispenser: the dispenser, thanks to the airless technology that eliminates the air inside, allows optimal conservation of the freshly prepared product and allows the patients to always have with them, in a practical and convenient way, the whitening gel to use with their aligner/tray. TAILOR Airless can contain 9-10ml of whitening gel that the patient will use according to the time and method of use instructions provided by the professional; furthermore, the dispenser is equipped with a practical spout that helps the patient to dose the whitening gel in a precise and calibrated way, while the spatula allows the patient to easily distribute the product in the aligner/mask.

The package contains everything you need to create and deliver your own teeth whitening treatment to 4 PATIENTS:

- 4 White Beauty Professional TAILOR SOLUTION syringes + 4 White Beauty Professional TAILOR CARBAMIDE syringes
- 4 Luer-lock connectors
- 4 White Beauty Professional TAILOR Airless dispenser
- 4 spatulas
- 4 White Beauty Professional WHITE CARD and 4 adhesive certifications that will indicate the product batch and the preparation date.

HOW TO USE (it is advisable to view the preparation video by scanning the QR-code)

- Before proceeding with the procedure, wear safety devices (gloves, mask and glasses). The preparation procedure could be carried out in front of the patient to enhance the properties of the product; in this case, have the patient also use the safety devices.
- Remove the TAILOR CARBAMIDE syringe from the package, unscrew the cap and insert the connector.
- Remove the TAILOR SOLUTION syringe from the package, unscrew the cap and screw the syringe onto the connector to connect the two syringes together.
- Once the syringes are connected, press the piston of the TAILOR SOLUTION syringe, completely transferring the liquid into the TAILOR CARBAMIDE syringe.
At this point the liquid will start to wet the powder: leave it to act for at least a minute without mixing and slightly rotating the syringe to allow the liquid to wet all the powder.
- At the end of the previous phase, during which a cold chemical reaction was activated, start mixing by pressing the pistons alternately for about twenty times until the gel becomes homogeneous.
If you encounter resistance to the pressure of the piston, do not continue to insist and push the opposite piston continuing to mix until the gel becomes fluid between one syringe and the other.
- Once you have poured all the whitening product into a single syringe, remove the empty one and pour the gel into the TAILOR Airless dispenser.
- Once the dispenser has been filled with gel, place it on a stable surface, take the spout (without removing the cap) and place it on the dispenser: to complete the closure, hold the dispenser and use the palm of the other hand to press firmly until the dispenser is completely inserted.
- Remove the cap and press until the product comes out; this removes any air present inside the dispenser.
- Fill in the White Beauty Professional WHITE CARD with the data of the product just prepared.
- The teeth whitening product is then ready to be given to the patient. Remember to give them detailed instructions for use and advice on how to maintain the results of their whitening treatment for longer!

Advice for the patient: at the end of the treatment, rinse both the mouth and the mask with water to remove any residue.

WARNINGS: Limit unnecessary personal contact; avoid contact with eyes; wear adequate protective equipment. In case of contact with eyes, rinse immediately with plenty of water. If irritation persists, consult a doctor. Store in a cool, well-ventilated area, away from heat, sparks, flames and sunlight. TAILOR 10: contains 10% carbamide peroxide. TAILOR 16: contains 16% carbamide peroxide.

PRODUCT FOR PROFESSIONAL USE - The first use of the product must be under the supervision of a professional. The European Cosmetics Regulation prescribes not to use professional whitening products on people under the age of 18. However, the use of the product is at the discretion of the professional following clinical evaluation.

Produced by: MIROMED GROUP SA - Corso San Gottardo 54A - 6830 Chiasso - SWITZERLAND - www.miromedgroup.ch
MIROMED S.r.l. - info@miromed.it - Viale De Gasperi 17 - 20045 Lainate (MI) ITALY
MIROMED S.r.l. licensee MENTADENT PROFESSIONAL

Remove the TAILOR CARBAMIDE syringe from the package, unscrew the cap and insert the connector.

Remove the TAILOR SOLUTION syringe from the package, unscrew the cap and screw the syringe onto the connector to connect the two syringes together.

Once the syringes are connected, press the piston of the TAILOR SOLUTION syringe, completely transferring the liquid into the TAILOR CARBAMIDE syringe.

At this point the liquid will begin to wet the powder: leave it to act for at least a minute without mixing and rotating the syringe slightly to allow the liquid to wet all the powder.

At the end of the previous phase, during which a cold chemical reaction was activated, begin mixing by pressing the pistons alternately for about twenty times until the gel becomes homogeneous. If you encounter resistance to the pressure of the piston, do not continue to insist and push the opposite piston continuing to mix until the gel becomes fluid between one syringe and the other.

Once all the whitening product has been poured into a single syringe, remove the empty one and transfer the gel into the TAILOR Airless dispenser.


Once the dispenser has been filled with gel, place it on a stable surface, take the spout (without removing the cap) and place it on the dispenser: to complete the closure, hold the dispenser and use the palm of the other hand to press firmly until the dispenser is completely inserted.

Remove the cap and press until the product comes out; this removes any air present inside the dispenser.

Fill in the White Beauty Professional WHITE CARD with the data of the product you have just prepared.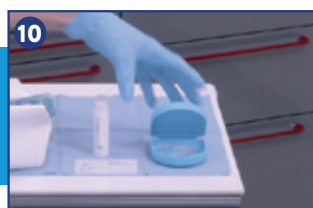

The product is ready to be delivered to the patient. Remember to give him detailed instructions for use and advice to maintain the results of the whitening treatment longer!

Insert Tailor Airless into the pouch together with the spatula, White Now Revitalize toothpaste and the Mentadent Professional Sensibilità toothbrush and give it to the patient.

Scan to view
the procedure video

Advice for the patient:
At the end of the treatment, rinse both the mouth and the mask with water to remove any residue.