WHITE[®] | BEAUTY PROFESSIONAL

Genius Laser

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PROFESSIONAL DENTAL WHITENING TREATMENT based on Hydrogen Peroxide

White Beauty Professional GENIUS 38% is used to whiten teeth with discoloration. According to the Medical Device Directive 93/42/EEC the product is classified as a class I medical device.

Its use is permitted only in suitable places (dental offices) and only to doctors and specialists regularly authorized to practice.

COMPOSITION:

Liquid: Hydrogen peroxide. Solution with 38% dilution in distilled water, co formulant.

Anhydrous products: Gelling agent, E 162, potassium nitrate, co formulant.

ADVICE ON USE:

Follow the instructions for use in order to use the product correctly.

The duration of the applications and the relative suitability of the patient to be subjected to this type of treatment always remains to be assessed by the professional following the clinical evaluation.

INDICATIONS: It can be used for a single tooth or several teeth with discoloration or devitalized at the same time.

The discoloration may be caused by: medicines, fluorosis, food stains and ageing. The product must not be used on patients with verified allergies to the product components. It is recommended to complete the whitening treatment at least two weeks before the placement of any new restorations. Existing restorations, on which the product do not work, may not match the lighter color of the teeth after the whitening treatment and may need to be replaced.

CONTRAINDICATIONS: Do not use the product on patients during pregnancy or when breastfeeding, on children, heavy smokers or heavy drinkers, patients allergic to any of the product components, on teeth with the exposed root, on primary and secondary untreated caries, on crowns and bridges with marginal cracks.

SIDE EFFECTS: There may be a temporary teeth sensitivity; these symptoms usually disappear when the treatment is finished or interrupted.

WARNING: White Beauty Professional GENIUS LASER contains hydrogen peroxide which is an irritant. It is recommended the use of gloves, surgical masks and protective goggles. Avoid the contact of the gel with the skin and mucosa, in case of contact rinse with water the area immediately and thoroughly. Consult your doctor if the irritation persists for several hours. In case of contact with eyes, consult immediately the doctor. Make the complete treatment maximum 2 times per patient in an year.

In case of swallowing significant quantity of whitening gel, drink plenty of water and consult the doctor.

The spatula used for mixing and application is disposable and should not be used for different patients in order to avoid possible cross contamination.

Avoid contact of the product with alkaline substances as they are not compatible.

Warning: the product ignites at 240° C.

Before using the product, the doctor must check the integrity of the packaging. Dispose the used product according to the local and governmental authorities regulations. The product cannot be used for purposes other than that for which it was designed.

NOTE: Keep out of children's reach. The product must be used in oral treatment according with the specific instructions for use.

STORAGE: The product should be kept in room temperature. Avoid direct light exposure and heat. The expiry date refers to the properly stored product. Do not use after the reported date is expired.

APPLICATION

METHOD TO BE PERFORMED ONLY AT THE DENTAL OFFICE

Before to start the whitening treatment procedure, the patient should be informed about the complete procedure and has to sign the informed consent.

The professional in the office takes the patient's medical history and notes the indication for treatment via the color scale or the WBP Card. Make sure that the teeth to be treated are absolutely clean and proceed with the color setting. In case of proven dentinal sensitivity, the preventive application of the treatment based on micronized hydroxyapatite is recommended.


Lift the lip using a mouth opener. Protect the gums using White Beauty Professional QUICK MASK BLUE (after applying light cure as per instructions for use).

Before mixing the pink gel product in the syringe, wear gloves, a mask and protective glasses. Remove the GENIUS LASER syringe from the package, unscrew the cap and insert the connector.

Unscrew the cap of GENIUS POWDER and connect the syringe to the connector.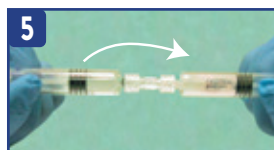

Once both syringes have been connected to the connector, vigorously press the piston of the GENIUS LASER syringe, completely pouring the liquid into the GENIUS POWDER syringe.

Continue mixing the product by pressing the syringe pistons alternately and vigorously 3/4 times.

Once a homogeneous gel has been obtained, transfer all the product obtained into the GENIUS LASER syringe alone and remove the connector.

Insert the tip and compress the homogeneous pink gel until the air inside the syringe escapes.

Apply the pink whitening gel directly to the teeth and activate it using the laser tooth by tooth or arch based on the type of laser used. The duration of the treatment depends on the type of laser used.

Aspirate the product and possibly repeat a second application step (do not use water). **The procedure can be repeated up to two steps** (always following the instructions of the professional and the type of laser used).

Perform color sampling to check the results obtained. Remove the liquid dam used to protect the gums.

In case of sensibility after treatment it is suggested to repeat the application of the treatment with micronized hydroxyapatite.

USAGE NOTES:

- The duration of the treatment and the possible second step are determined by the type of laser used: refer to the instructions given by the manufacturer.
- For the patients who teeth sensibility highly reacts, reduce the contact time.
- Control the status of the teeth and the gingiva during the treatment.
- Once mixed the 2 syringes the product should be used within max. 60-90 minutes.