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PROFESSIONAL WHITENING TREATMENT based on Hydrogen Peroxide

White Beauty Professional GENIUS 38% is intended exclusively for professional use in the dental practice. It is applied by the dental professional for whitening one or more pathologically discolored teeth. The whitening action is effective both with and without the use of an LED lamp (if used, the lamp accelerates the action of the whitening gel). The medical device is used for rapid whitening, as indicated by the clinician, of pathologically discolored vital and non-vital teeth whose color cannot be restored to its natural or normal shade through professional cleaning. The medical device is also used on non-vital teeth, including in-office intracoronary whitening, and represents an alternative and conservative method for whitening dark and discolored teeth that cause a deterioration in the patient's oral health-related quality of life due to the abnormal tooth color caused by predisposing factors such as diseases, traumatic or iatrogenic injuries, congenital, systemic, metabolic or pharmacological factors. These may include, by way of example and without limitation: high bilirubin levels, tetracycline and adult minocycline staining, porphyria or jaundice, fetal erythroblastosis, high fluoride intake during dental development, moderate to severe dental fluorosis, non-vital teeth (pulp necrosis), genetic or age-related (degenerative) discolorations, and heavy food deposits with penetration of extrinsic colorants into the dental substance.

COMPOSITION:

Liquid base: Hydrogen peroxide (38% dilution in distilled water).
Anhydrous products: Gelling agents, potassium nitrate, EDTA, C.I. 42090.

DIRECTIONS FOR USE: Follow the instructions for use to ensure correct use of the product.

The duration of the applications and the patient's suitability for this type of treatment must always be assessed by the professional following the clinical evaluation.

INDICATIONS: A single tooth or several teeth affected by discoloration or devitalized teeth may be whitened at the same time. The product must not be used in patients with confirmed allergies to any of the components. It is recommended to complete the whitening gel treatment at least two weeks before placing new aesthetic restorations. Existing restorations, on which the gel has no effect, may not match the lighter tooth shade after treatment and may need to be replaced.

CONTRAINDICATIONS: Do not use the product on patients allergic to any component of the product, during pregnancy or breastfeeding, or on anesthetized patients. Use of the product is at the discretion of the professional following the clinical evaluation. Use should also be avoided in cases of inflamed gums, teeth with exposed cervical areas, untreated primary or secondary caries, and crowns or bridges with marginal gaps.

SIDE EFFECTS: Temporary tooth sensitivity may occur; these symptoms usually disappear at the end of the treatment or when treatment is discontinued.

WARNINGS: White Beauty Professional GENIUS 38% contains hydrogen peroxide, which is an irritant. The use of gloves, masks and protective eyewear is recommended.

Avoid contact of the gel with skin and mucous membranes; in case of contact, immediately rinse the affected parts of the body thoroughly with water.

If irritation persists even after several hours, consult a physician. In case of contact with eyes, consult a physician immediately. The spatula is single-use and must not be used for different patients in order to avoid possible cross-contamination. Avoid contact of the product with alkaline substances, with which it is incompatible. Warning: the product ignites at 204°C.

Before using the product, medical personnel must check the integrity of the product packaging. Disposal of used, damaged or expired product must be carried out in accordance with the rules on disposal of dental waste contained in Italian Legislative Decree of 05/02/97, as subsequently amended and supplemented. The product must not be used for purposes other than those

	Symbol identifying the MANUFACTURER.
	Symbol identifying the Authorized Representative located in the European Community.
	Symbol for medical devices.
	Expiry date symbol. This symbol shall be accompanied by the date by which the device must be used.
	Batch code symbol. This symbol shall be accompanied by the batch number of the product.

for which it was designed. In the event of a serious incident occurring in relation to this product, report the incident to the manufacturer and to the competent authority of the Member State in which the serious incident occurred, quoting the batch number on the package.

NOTES: Keep out of reach of children. To be used exclusively in the oral cavity according to the specific instructions for use.

METHOD OF USE: DIRECT METHOD TO BE PERFORMED ONLY IN THE DENTAL PRACTICE

Before proceeding with the dental whitening treatment, the patient must sign the informed consent form after receiving all information about the procedure.

- The professional in the practice takes the patient's medical history and determines the indication for treatment using the shade guide or the WBP Card. Ensure that the teeth to be treated are absolutely clean and proceed with shade recording.
- In the event of confirmed dental sensitivity, preventive application of the treatment based on micronized hydroxyapatite is recommended.
- Lift the lip using a mouth opener. Protect the gingiva by applying the White Beauty Professional QUICK MASK BLUE liquid dam (at the end of application, light-cure it according to the product instructions).
- Before mixing the blue gel product in the syringe, wear gloves, a mask and protective eyewear. Remove the GENIUS 38% syringe from the package, unscrew the cap and screw on the connector.
- Unscrew the cap of GENIUS POWDER and connect the syringe to the connector.
- Once both syringes are connected to the connector, firmly press the plunger of the GENIUS 38% syringe, completely transferring the liquid into the GENIUS POWDER syringe.
- Continue mixing the product by pressing the syringe plungers alternately and firmly 3/4 times.
- Once a homogeneous gel has been obtained, transfer all the product obtained into the GENIUS 38% syringe only and remove the connector.
- Insert the tip and compress the homogeneous blue gel until the air present inside the syringe is expelled.
- Apply the blue whitening gel directly onto the teeth. The whitening action is effective both with and without the use of an LED lamp (if used, the lamp accelerates the action of the whitening gel). The duration (10/15 minutes per step) and the number of treatments (maximum 2) remain at the professional's discretion based on the patient's clinical evaluation.
- Aspirate the product and, if necessary, repeat a second application step (do not use water).

The procedure may be repeated up to two steps (always following the professional's instructions).

- Record the shade to verify the results obtained.
- Remove the liquid dam used to protect the gums.
- In the event of post-treatment dental sensitivity, repeat the application of the product based on micronized hydroxyapatite.

NOTES FOR USE:

- The procedure may be repeated up to 2 steps.
- For patients who react with excessive tooth sensitivity, reduce the contact time.
- Check the condition of the teeth and gums during treatment.
- Once the two syringes have been mixed, the product may be used for a maximum time of 60-90 minutes.

STORAGE: Store the product at room temperature, protecting the packaging from heat and exposure to direct sunlight. The expiry date refers to the product when perfectly stored. Absolutely do not use after the expiry date shown on the package.

	CE conformity marking with the identification number of the Notified Body that issues the CE marking.
	Symbol indicating that the product is single-use and non-reusable.
	Symbol indicating to consult the IFU or the IFU in electronic format.
	Caution symbol.

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MIROMED S.r.l. - Viale De Gasperi 17 - 20045 Lainate (MI) ITALY - info@miromed.it